

How to Attach Osto-EZ-Vent[®]



PREP: Attach OEV™ nearest the top on the outside of the non-stoma side of the pouch. If the pouch has a fabric or mesh cover/comfort layer, cut a hole in that layer to expose the plastic pouch wall below. Do not attach the OEV™ on top of a filter (if applicable). Store unused OEV™ at 72° F (24°C).



1) Moisten cotton pad with 91% isopropyl alcohol. Rub plastic pouch wall until the gloss of the pouch surface is removed. Let dry.



2) Peel away the green/white backing from the OEV™ to reveal the adhesive. Do not touch the adhesive.




3) Place the OEV™ horizontally on the cleaned area. Press firmly around the circular base of the OEV™ as shown at left. Set pouch aside for 24 hours to allow adhesive to securely bond to your pouch.



4) Pull down the side of pouch where the OEV™ is attached so the adhesive side of the OEV™ is opposite the stoma opening of your pouch (for one-piece pouches, cut your stoma opening first).



5) With the narrow end of scissors, puncture pouch wall behind the OEV™. Twist the blade back and forth to remove excess pouch material. The hole should be the size of the OEV™ opening.

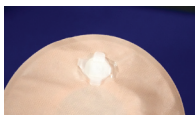

6) With the hole created, flatten pouch to its original position and close the OEV™ as shown at left.



7) To open the OEV™, pinch DOWN on the OEV™ Glides (tabs). The Glides will push against the circular base of the OEV™ to lift the plug up and out.

For more information on How to Attach including a video walkthrough visit kemOnline.com.



Enterprises, Inc.

© 2023 Printed in U.S.A.
OEV™ Manufactured in USA

OstoEZVent.com

info@kemOnline.com

888-562-8802